

Patient Care Instructions



As easy as 1, 2, 3

- 1 Rinse after every meal** – both your denture and mouth, and use a soft toothbrush to clean denture.
- 2 Soak for 15 minutes each day in a commercial cleanser for thorough cleaning** – and when not in your mouth, soak in clean, clear water.
- 3 Have regular scheduled appointments with your dental professional** to ensure fit, comfort and oral tissue check.

Dear Denture Wearer: Congratulations!

Your denture has been fitted with an EasySoft liner™ – a soft, comfortable, cushioning material that is attached to your hard, pink acrylic denture base. Your EasySoft Cushion Comfort liner™ will ensure the most comfortable fit possible, especially while chewing. For the first 24 hours after fitting of your EasySoft liner™, do not brush your denture with a toothbrush. Gently rinse it under cool running water.

How long will my EasySoft liner™ last? Your EasySoft liner™ will last longer with careful daily care. Ask your dental professional when is the best time to replace your EasySoft liner™. The liner should be checked by your dental professional at your regularly scheduled recall/recare appointment; we recommend at least once a year, during your annual check up appointment.

How can I best care for my EasySoft liner™?

- Rinse well under cool to warm running water.
- Gently clean with a very soft toothbrush and non-abrasive toothpaste. Do not scrub.
- Do not use commercial denture brushes – they are too coarse.
- Rinse again before returning to the mouth.
- When not in your mouth (e.g. overnight), your denture should be soaked in plain, warm water.

Some cleansing products may damage your EasySoft liner™.

- Do not use commercial denture soaking solution any longer than 15 minutes each day.
- Do not use bleach or vinegar or other harsh products.
- Consult your dental professional before using any cleanser to avoid permanent damage.
- Do not place in microwave or dishwasher to clean.

Can my EasySoft liner™ be adjusted if I have a sore spot? Yes, however, the adjustment requires specialized instruments and techniques, so it's important that you do not attempt to adjust the EasySoft liner™ yourself. Contact your dental professional.

Will my EasySoft liner™ change colour over time? Yes, it may. If you drink coffee, tea, dark juices or red wine, or if you smoke, your liner will darken over time. If you use cleaning material too long (*more than the recommended 15 minutes daily*) your liner will lighten in colour. The colour change does not alter the softness, fit or bonding strength of your EasySoft liner™.

YOUR NEXT APPOINTMENT:

Karlin
DENTAL DISTRIBUTORS INC